



REACTHEALTH

Download the **FREE**
3B MASK FIT APP
from the App Store
or Google Play to
your smart device.



QR CODE APP INSTRUCTIONS

✿ If you have a **Luna II Device (CPAP LG2000, Auto- PAP LG2A00)**, while in the STAND-BY MODE, briefly press the ALARM SILENCE button located to the left of the selector knob.

If you have a **Luna G3 Device (CPAP LG3500, Auto-PAP LG3600, Bilevel 25A LG3700 or Bilevel ST 30Vt LG3800)**, turn the knob until Report is highlighted in BLUE, push the knob, and select iCODE QR/QR+.

✿ You will now see the iCode, iCode QR and iCode QR+ screens. Select iCode QR+ by turning the selector knob clockwise or counterclockwise. Once selected, the corresponding QR+ code will appear in the display window. Using your smartphone or smart device, open the 3B Mask Fit App. Once opened, select the iCode QR+ option.

✿ Center the QR Code within the square on your smart device. Your device will now begin to SCAN for the QR Code.

✿ Once scanned, you will see a summary of the scanned data and have an option to review the detail. If applicable, the sleep data will also be automatically transmitted and uploaded into the patient's file associated with the patient's serial number and PIN.

✿ To exit from the iCode QR+ screen on the **Luna II**, turn the selector knob and choose BACK and press the knob. For **G3 devices**, press the HOME button twice.

✿ This app only supports the QR+ option.



REACT HEALTH, FORMERLY KNOWN AS 3B MEDICAL, INC.