



# MODEM PINS QUICK GUIDE

## LUNA II & LUNA G3 PAPS

In order to facilitate remote device setting changes each PAP device equipped with a modem has a PIN number. This PIN number is unique to the device and modem.

### INSTRUCTIONS FOR LUNA II

- 1 Plug the modem into the USB port on the back of the device
- 2 Power up the Luna II device. Once booted up, hold down the "RAMP" button until the Patient Comfort menu appears
- 3 Press the KNOB once to enter the menu
- 4 Turn the KNOB clockwise and scroll down to "ABOUT"
- 5 Press the KNOB once to enter the "ABOUT" menu  
The last line of information displayed will be the unique PIN
- 6 Enter the PIN into the iCodeConnect patient database to enable  
REMOTE DEVICE SETTING changes

### INSTRUCTIONS FOR LUNA G3

- 1 Power up the G3 device. Turn the KNOB until SETTINGS is highlighted
- 2 Push the KNOB once to enter the Patient Comfort menu
- 3 Turn the KNOB clockwise until you reach the last option on the second screen entitled "ABOUT"
- 4 Push the KNOB once to enter the "ABOUT" menu  
The fifth line of information ("ABOUT" halfway down the page) is the PIN
- 5 Enter the PIN into the iCodeConnect patient database to enable "REMOTE DEVICE SETTING" changes

\*NOTE: changing either modem or PAP device will result in a new unique PIN. Further information "ABOUT" the iCodeConnect compliance database for 3B Medical, Inc. PAP devices can be found at [www.iCodeConnect.com](http://www.iCodeConnect.com) or by emailing [info@3BProducts.com](mailto:info@3BProducts.com)