



iCODECONNECT: REGISTERING A NEW ACCOUNT FOR PHYSICIANS AND SLEEP LABS

Physicians often want to have access to a patient for monitoring and reporting on PAP efficacy. In the iCodeConnect system it is currently possible to connect ONE entity to a patients record. For Physician groups that share patient access it is requested that they register as a group.

For example, instead of registering as Dr. John Doe, they register as the Doe Sleep Center. This serves to allow an easier way for the DME customer to find and tag the physician group. Occasionally, some groups are not able to share a log in (often due to Health System Security Regulations) in these instances the DME can add the physicians as a Subsidiary Provider.

Important Note: If the DME adds the Physician as a subsidiary provider vs. adding as a Physician, the Physician will have access to editing details.

1

Go to www.icodeconnect.com

2

Register for a Physician Account as shown below:

NOTE: If you are registering as a sleep lab you will place the name in both the first and last name boxes

Example: First: Doe Sleep Lab
Last: Doe Sleep Lab

3

Register with your information as shown below. Items with * are required fields



iCODECONNECT: REGISTERING A NEW ACCOUNT FOR PHYSICIANS AND SLEEP LABS (Continued)

4 Log in to see patients that have been assigned to you

Physician Accounts are able to:

- Download SD card reports
- Input 16-digit iCodes
- Run Comprehensive Therapy Reports
- Run Summary Therapy reports
- Push device settings changes for patients with a modem

5 If patients are assigned to you, you a Patient List will populate as shown below

Patient ID	Machine Serial	Machine PIN	City	State	Clinician	Therapy start	Watch Group	Compliance (%)
15824317	E2316807046	18885	Leschfort	MIN	Auer, Gilbert	12/27/2016		7%
48336569	E2316807046	25541	Port Lillian	KY	Auer, Gilbert	12/27/2016		100%
51255136	E2316804124	12979	Steubersmouth	WA	Auer, Gilbert	11/26/2016		57%
73804689	E2316809219	16527	Lake Liliana	NI	Auer, Gilbert	03/07/2017		99%
5771194	E2316809227	20413	Marvinberg	FL	Auer, Gilbert	04/20/2017		No data
81536158	E2316810038	14021	Rodriguezland	AK	Auer, Gilbert	05/15/2017		0%
55999867	E2316809155	17046	North Floydport	VA	Auer, Gilbert	02/14/2017		No data
74656374	E2316804029	22176	West Elita	NV	Auer, Gilbert	12/29/2016		37%
34208478	E2316804132	18835	South Ibrahim	OH	Auer, Gilbert	12/16/2016		100%
95932263	E2316809247	16560	Elenorfurt	UT	Auer, Gilbert	03/17/2017		7%
48890712	E2316807193	22187	Port Saige	KS	Auer, Gilbert	05/11/2017		No data
72280094	E2316807137	21693	Krystalport	NM	Auer, Gilbert	05/05/2017		0%
44355916	E2316809139	21399	Powlowskilburgh	IL	Auer, Gilbert	02/28/2017		73%
66759366	E2116801049	13985	Rampilton	VT	Auer, Gilbert	11/14/2016		43%
		23703	West Keara	IN	Auer, Gilbert	01/31/2017		47%