



MODEM PINS QUICK GUIDE

LUNA II & LUNA G3 PAPS

In order to facilitate remote device setting changes each PAP device equipped with a modem has a PIN number. This PIN number is unique to the device and modem.

INSTRUCTIONS FOR LUNA II

- 1 Plug the modem into the USB port on the back of the device
- 2 Power up the Luna II device. Once booted up, hold down the "RAMP" button until the Patient Comfort menu appears
- 3 Press the KNOB once to enter the menu
- 4 Turn the KNOB clockwise and scroll down to "ABOUT"
- 5 Press the KNOB once to enter the "ABOUT" menu
The last line of information displayed will be the unique PIN
- 6 Enter the PIN into the iCodeConnect patient database to enable
REMOTE DEVICE SETTING changes

INSTRUCTIONS FOR LUNA G3

- 1 Power up the G3 device. Turn the KNOB until SETTINGS is highlighted
- 2 Push the KNOB once to enter the Patient Comfort menu
- 3 Turn the KNOB clockwise until you reach the last option on the second screen entitled "ABOUT"
- 4 Push the KNOB once to enter the "ABOUT" menu
The fifth line of information ("ABOUT" halfway down the page) is the PIN
- 5 Enter the PIN into the iCodeConnect patient database to enable "REMOTE DEVICE SETTING" changes

*NOTE: changing either modem or PAP device will result in a new unique PIN. Further information "ABOUT" the iCodeConnect compliance database for React Health PAP devices can be found at www.iCodeConnect.com or by emailing info@reacthealth.com.